



# PATIENT MANUAL

**NTX150**

TONIC MOTOR ACTIVATION SYSTEM



# TABLE OF CONTENTS

<b>INTRODUCTION</b>	<b>3</b>		
<b>CHAPTER 1—SAFETY INFORMATION</b>	<b>4</b>		
• Indications for use	4		
• Important safety information	4		
• Contraindications	4		
• Warnings	4		
• Precautions	5		
• Adverse events	6		
• Symbols	6		
• Clinical study: RLS symptoms and sleep quality	7		
<b>CHAPTER 2—SYSTEM OVERVIEW</b>	<b>8</b>		
• System components	8		
• Therapy control features	9		
• Status Lights	9		
• Glossary	11		
<b>CHAPTER 3—CARING FOR YOUR DEVICE</b>	<b>12</b>		
• Charging the system	12		
• Caring for the NTX150 TOMAC system	13		
• Removing the e-module	13		
• Storage instructions	13		
• Instructions for safe disposal	13		
<b>CHAPTER 4—WHEN TO USE YOUR DEVICE</b>	<b>14</b>		
• Overview	14		
• General timing instructions	14		
• Examples of common times to use the device	15		
<b>CHAPTER 5—TREATMENT</b>	<b>16</b>		
• Inserting the e-module	16		
• Placement of Charge-Dispersing Interface (CDI) on the device	16		
• Finding the treatment target	17		
• Putting on your devices	18		
• Starting a treatment	20		
• Adjusting treatment level during a treatment session	21		
• Stopping a treatment	21		
• During a treatment session	21		
• After a treatment session	21		
<b>CHAPTER 6—TROUBLESHOOTING</b>	<b>22</b>		
• Patient troubleshooting	22		
<b>PATENT</b>	<b>23</b>		
<b>REFERENCES</b>	<b>23</b>		
<b>APPENDIX A—GUIDANCE AND MANUFACTURER’S DECLARATION</b>	<b>24</b>		
<b>APPENDIX B—SYSTEM SPECIFICATIONS</b>	<b>27</b>		

## INTRODUCTION

**This manual is for people who have received the NTX150 Tonic Motor Activation (TOMAC) System to treat symptoms of Restless Legs Syndrome (RLS). Each person is unique. Always talk to your doctor if you have questions about your condition. This manual explains how to use the system.**

The appendix provides more information about the device. Carefully read all instructions before using the NTX150 TOMAC System. Follow all the safety information in this manual. Failure to do so may result in the possibility of injury, less effective treatment, or damage to the device.

For questions contact Noctrix® Customer Service at 1-866-EASE-RLS (1-866-3273-757).





# CHAPTER 1—SAFETY INFORMATION

## Indications for use

The NTX150 Tonic Motor Activation (TOMAC) System is intended to reduce symptoms of primary moderate-severe Restless Leg Syndrome and improve sleep quality in adults refractory to medications.

## Important safety information

The following symbols are found in this manual:

Symbol	Definition
 <b>WARNING!</b>	A warning indicates a situation which, if not avoided, could result in death or serious injury.
 <b>PRECAUTION!</b>	A precaution indicates a situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

## Contraindications

The NTX150 Tonic Motor Activation (TOMAC) System is contraindicated for use in patients with the following:

- Diagnosis of epilepsy or other seizure disorder.
- Active medical device implant anywhere in the body, including but not limited to pacemakers, spinal cord stimulators, deep brain stimulators.
- Metal implant in the leg at the therapy site (not including knee replacements).
- Known allergy to device materials (or severe previous reaction to medical adhesives or bandages).
- Cellulitis, open sores, or injury at or near the location of therapy device application.

The device cannot be used while driving, operating machinery, or during any activity in which electrical stimulation can put the patient at risk of injury.

## Warnings

Warning	Description
<b>Electrical</b>	<p>The NTX150 TOMAC System is not waterproof. DO NOT expose to water.</p> <p>There are no user-serviceable parts in or on the NTX150 Tonic Motor Activation (TOMAC) System. Do not attempt to service or modify the device beyond the basic cleaning instructions in this manual.</p> <p>Operation of this equipment in electrical environments discussed in Appendix A may result in improper operation. Special precautions are needed regarding:</p> <ul style="list-style-type: none"><li>• The NTX150 TOMAC System is not intended for use in a Magnetic Resonance Environment.</li><li>• Operation in close proximity to a shortwave or microwave medical electrical equipment.</li></ul> <p>The long-term effects of electrical stimulation are unknown.</p> <p>Since the effects of stimulation of the brain are unknown, stimulation should not be applied across the head, and electrodes should not be placed on opposite sides of the head. Stimulation should not be applied directly on the eyes, covering the mouth, on the front of the neck, or from electrodes placed on the chest and the upper back or crossing over the heart.</p> <p>The safety of electrical stimulation during pregnancy has not been established.</p> <p>Patients with suspected or diagnosed heart disease should follow precautions recommended by their physicians.</p> <p>Please do not use it on the area if the skin is not clean or not easy to reach.</p>

Warning	Description
<b>NTX150 TOMAC System Parts</b>	<p>The NTX150 Tonic Motor Activation System is intended for use only with the provided components. Substituting different components for those supplied by Noctrix Health, Inc. may damage the device and/or create a safety hazard.</p> <p>Avoid using the device next to, or stacked with, other equipment.</p> <p>Prior to use, inspect the product package for signs of damage or tampering. If damaged, do not use.</p> <p>Do not use if you are allergic to acrylic or adhesives.</p> <p>Do not connect the USB charging cable while wearing and/or using the device.</p> <p>Application of electrodes near the chest may increase the risk of cardiac fibrillation.</p> <p>Accessory cables represent a strangulation hazard.</p> <p>Keep out of reach of pets and children.</p>
<b>Charge-Dispersing Interfaces (CDIs)</b>	<p>Do not place Charge-Dispersing Interfaces (CDIs) over open sores or an acute injury.</p> <p>Stop use if skin irritation develops at or around the therapy site. Remove the system and consult your health care professional.</p>

## Precautions

Precaution	Description
<b>NTX150 TOMAC System</b>	<p>Carefully read all instructions before using the NTX150 TOMAC System. Follow all contraindications, warnings and cautions in this manual. Failure to do so may result in possible injury, less effective treatment, or damage to the device.</p> <p>The NTX150 TOMAC System should not be exposed to water.</p> <p>Position the device so it is easy to disconnect the charging cable.</p> <p>Note that premature onset of RLS symptoms while wearing the device may occur for some people as this device may interfere with voluntary leg movements used to relieve RLS symptoms, thus leading to a temporary increase in RLS symptoms. In some cases, this device may lead to a temporary increase in RLS symptoms during active stimulation for other reasons. This risk may be reduced by adjusting the stimulation intensity and changing the time of day when the device is used.</p> <p>Operation of this equipment in electrical environments discussed in Appendix A may result in improper operation. Special precautions are needed regarding:</p> <ul style="list-style-type: none"> <li>• Electromagnetic compatibility (EMC).</li> <li>• Proximity to strong electrostatic discharges (ESD) that are <math>\geq 15</math> KeV in magnitude. If the system malfunctions after exposure to ESD as described above, the device reset procedure may be performed to resume normal operation.</li> <li>• Proximity to portable and mobile radio frequency (RF) communications equipment.</li> <li>• The NTX150 TOMAC System is not recommended for use in close proximity to RFID readers.</li> </ul>

# CHAPTER 1—SAFETY INFORMATION

## Adverse events

Adverse events, or side effects, are risks associated with the use of this device. There is a potential for the following side effects, which are typically mild to moderate and resolve over time:










- Administration Site Reaction (e.g. skin irritation).
- Discomfort.
- Temporary interference with sleep while wearing the device. For some, the device may interfere with preferred sleep positions.
- You should stop using the device and should consult with a physician if you experience adverse reactions from the device.










**Proper use of the device as described in the Instructions for Use (IFU) can help reduce or prevent the following complications:**

- Discomfort, paresthesia (tingling or prickling sensation), or otherwise irritating or uncomfortable sensations during treatment. This risk is reduced by adjusting the stimulation intensity.

## Symbols

The following symbols are associated with the (NTX150 TOMAC System):

Symbol	Definition
	Caution Electrical Output
	Do not use if you have been fitted with a demand style pacemaker, unless directed to do so by a healthcare provider
	Consult Instructions for Use
	Caution – Refer to manual for important safety information
	Catalog Number
	Batch Code
	CAUTION! Federal (US) law restricts this device to sale by or on order of a licensed healthcare practitioner.
	Unique Device Identifier
	Serial Number

Symbol	Definition
	Do not use if the package is damaged or open
	Date of Manufacture
	Country of Origin
	Single Patient Use Only
	Keep Dry
	Indicates the upper limit and lower limit of temperatures to which the medical device can be exposed
	Use-By Date
	CLASS II medical device
	Do not dispose of in a landfill

## Clinical study: RLS symptoms and sleep quality<sup>1</sup>

**A clinical trial with 133 patients with moderate to severe RLS was conducted. Patients were treated either with the NTX100 TOMAC System or a mock device that looked like the NTX100 TOMAC System but was inactive.**

Several clinical metrics were used to compare the two systems. In all metrics the NTX150 TOMAC System improved RLS more than the mock device.


Device-related side effects were typically mild and resolved rapidly with minimal or no follow-up.

The safety and efficacy of this device has not been studied in pregnant women. Contact your doctor if you would like more information about this clinical study.

## CHAPTER 2—SYSTEM OVERVIEW

### SYSTEM COMPONENTS


#### Compressive Conductive Garments (CCGs)

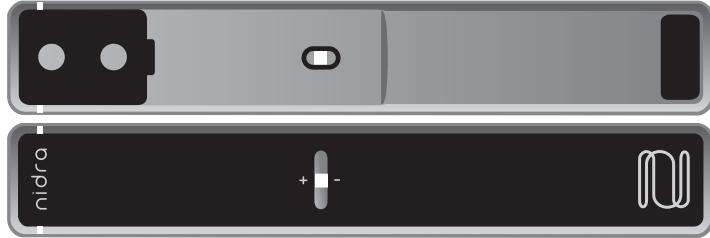

#### Charge Dispersing Interfaces (CDIs)




#### e-modules

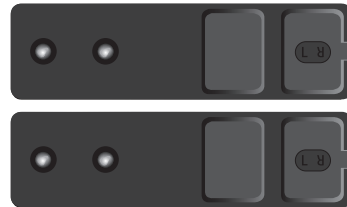

#### Charging accessories

- USB wall charger
- USB cables

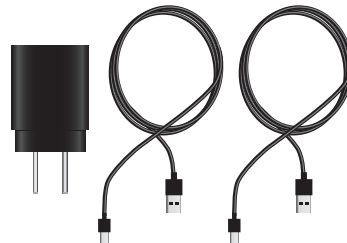

### The NTX150 Tonic Motor Activation (TOMAC) System consists of two Therapy Units, intended to be used on both legs.

The two units work separately but the mechanism of action is identical. An individual Personalized Activation Session is required and can result in different treatment settings for each leg. The device and buttons are mirrored on each leg (from left to right) to assist with placement on each leg.

The NTX150 TOMAC System consists of the distinct, non-sterile parts, listed below:

#### e-module

This module contains electronics and a rechargeable battery. The module is placed within the compressive conductive garment (CCG) and has buttons that turn the system ON/OFF, adjust treatment level, or read status.

#### Compressive Conductive Garment (CCG)

This conduction garment is worn on both legs (below the knee). The e-module sits within the CCG. When combined, these components are collectively referred to as the Therapy Unit. The Therapy Unit provides the TOMAC stimulation.

#### Charge Dispersing Interface (CDI)

This is a specially-designed reusable, adhesive electrode that allows current to flow from the device to the treatment target. The Charge-Dispersing Interface has a shelf life of 23 months.

#### Charging accessories

An AC wall adapter for US use (110V) with (2x) USB charging cables to recharge the battery on each device.

#### Carrying case

A carrying case (not pictured) is provided when water protection is needed.

# CHAPTER 2—SYSTEM OVERVIEW

## CONTROL FEATURES AND STATUS LIGHTS


### Therapy control features

Icon description	Icon	Action	Description
Turn On	+	Press and hold for three seconds	Turns on the Therapy Unit
Wake-up	+/-	Press both at the same time	Shows the treatment level lights and allows changing the treatment levels
Increase treatment level	+	Click once	Increases treatment by one level when device is awake
Decrease treatment level	-	Click once	Decreases treatment by one level when device is awake
Turn off	-	Click and hold for three seconds	Turns off the Therapy unit

### Indicator Lights


The Status Light is one colored light that can be either **ORANGE**, **GREEN**, or off.

The Treatment Level Lights are five white lights that can be either on or off.

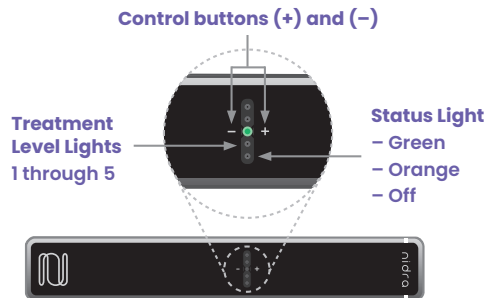


### Charging

When the device is plugged into the charger

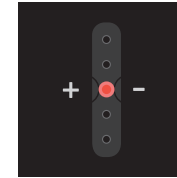


#### Orange Status Light:

Battery is charging but is not fully charged.

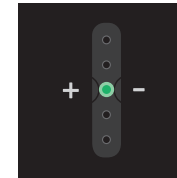

#### Green Status Light:

Battery is fully charged.

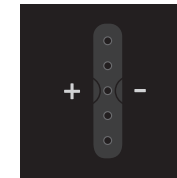

#### Status Light OFF:


Battery is not charging. Try to plug the device into the charger again or check the power supply to the outlet.

# CHAPTER 2—SYSTEM OVERVIEW


## STATUS LIGHTS

### Treatment

Ramp at the start of treatment (0–20 seconds)

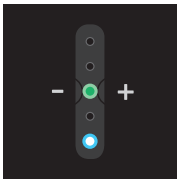


Treatment Level Lights (Left device)

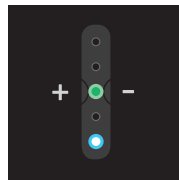


Treatment Level Lights (Right device)

**Blinking white Treatment Level Light and blinking green Status Light** appear for the first ~20 seconds of treatment: Device is on and ramping up to your treatment level. If these lights do not appear, the device was not turned on. Try turning the device on again.

Treatment level

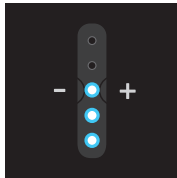

Treatment Level Lights (Left device)

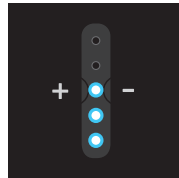


Treatment Level Lights (Right device)

For all of treatment, the **Treatment Level Lights** remain off unless the device is woken up by clicking the +/- buttons at the same time. The **number of Treatment Level lights** indicates your treatment level, from 1–5.

### Other

Charge status

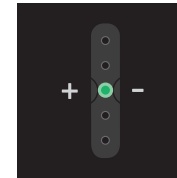


Charge status is displayed when the device is plugged-in and charging. If unplugged with therapy off, clicking the “+” button will display the charge status momentarily.

**Solid GREEN:** Typically enough charge for a full session.

**Solid ORANGE:** Low charge, but okay to proceed.

**Blinking ORANGE:** A session cannot start.

Reboot

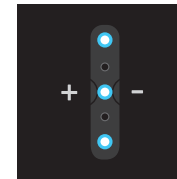

Rebooting the devices is performed by pushing the “-” button for 12 seconds. The three **Treatment Level Lights** at positions one, three, and five will confirm successful reboot.

Do not reboot unless instructed to do so.

Left/Right Therapy Unit Indicator




The L/R light is lit for 5 minutes after the device is unplugged and for 5 minutes after the “+” button is pressed. If the “L” is lit then it is the Left Therapy Unit. If the R is lit then it is the Right Therapy Unit.



# CHAPTER 2—SYSTEM OVERVIEW


## GLOSSARY

### Therapy Units (Left and Right)

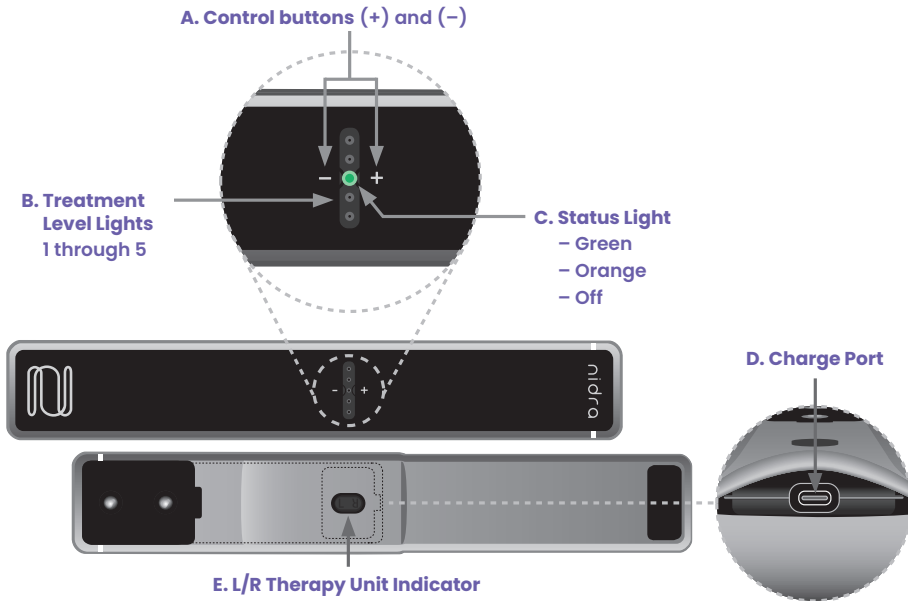


- A. Control buttons:** (+) and (-) buttons used to start/stop a session and adjust treatment level.
- B. Treatment Level Lights:** five lights which each can be white or off and indicates the treatment level.
- C. Status Light:** one light which can be **GREEN**, **ORANGE**, or off and indicates charging and battery status.
- D. Charge port:** opening in device for inserting the charging cable.
- E. L/R Therapy Unit Indicator Light:** Left or Right Therapy Unit.

### Charging accessories

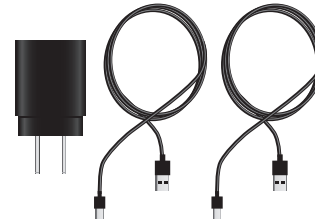

- Wall charger:** 120V Dual-USB charger that charges devices when attached to a charging cable.
- Charging cables:** USB connector that charges devices when attached to power outlet via the wall charger.

### Charge Dispersing Interface (CDI)

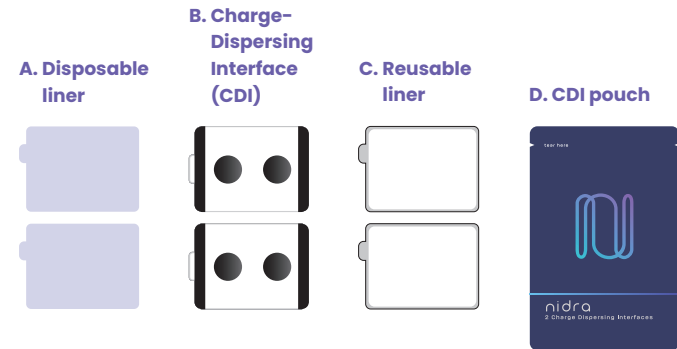

- A. Disposable liner:** blue see-through CDI liner.
- B. Charge-Dispersing Interface (CDI):** adhesive patch that attaches to the device and is replaced as directed.
- C. Reusable liner:** non-see-through CDI liner.
- D. CDI pouch:** container including two CDIs, one for each leg.

# CHAPTER 3—CARING FOR YOUR DEVICE

## IMPORTANT

Although the NTX150 Tonic Motor Activation (TOMAC) System arrives partly charged, it is recommended that the devices be FULLY CHARGED before using them for the first time.

You should charge both devices daily after each use and keep charging the devices until the next session (keep the devices on the charger). A full charge may take at least 2+ hours until the Status Light turns GREEN, depending on your therapy settings and usage.

For best results, replace the Charge-Dispersing Interfaces (CDIs) as directed.

## Charging the system

Please follow these steps to charge your Therapy Units. As noted in Section 9, the ORANGE light will come on once charging begins, and when charged, the light will change to GREEN.



**WARNING!**

Do not connect the USB charging cable while wearing and/or using the device.



**WARNING!**


The NTX150 TOMAC System is intended for use only with the provided parts. Using parts other than those supplied by Noctrix Health, Inc. may damage the device and/or create a safety hazard.

## NTX150 TOMAC System charging steps


1. Connect the male USB plugs into the female USB plug on the wall charger, and connect to a standard wall outlet.



2. Connect the male USB-C plug into the female USB-C port on the e-module.

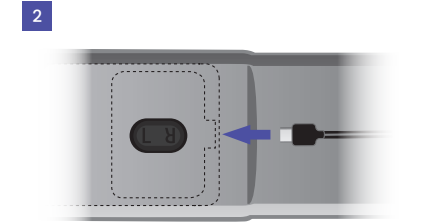

3. Once connected, the ORANGE Status Light should turn on, showing that the device is charging. You should charge both devices daily after each use and keep charging the devices until the next session (keep the devices on the charger). A full charge may take at least 2+ hours until the Status Light turns GREEN, depending on your therapy settings and usage.

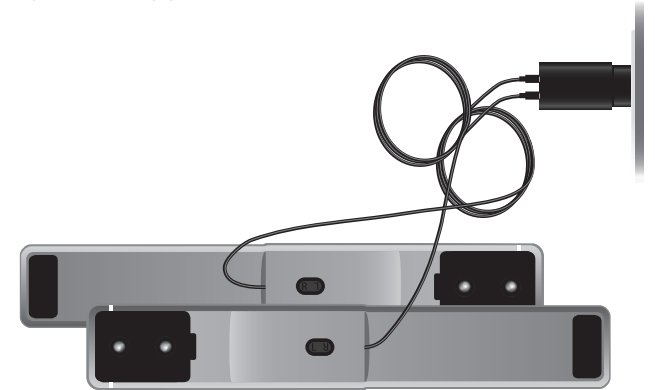

## CHAPTER 3—CARING FOR YOUR DEVICE

### Caring for the NTX150 TOMAC System



**WARNING!**

The NTX150 TOMAC System is not waterproof. Keep dry; DO NOT expose to water.

### Therapy Units

- The NTX150 TOMAC System devices are not waterproof and should not be placed in water.
- Attempts to wash the devices by hand, or in a laundry machine, will result in damage to the device.
- Be sure the device has been turned OFF before you start the cleaning.
- Only use a clean and dry cloth to remove dust as needed.
- Do not use soap, hand sanitizer, detergents, or other cleaners when cleaning the device.
- Do not use corrosive substances to clean the device or cables.

### Charge Dispersing Interfaces (CDIs)

- CDIs are not intended to be cleaned and should be disposed of as directed.
- CDIs are single patient use only and should not be shared.
- If the sticky portion of the CDI that contacts the leg loses its ability to stick, then use a new CDI.
- Place the reusable liner on the patient side of the CDI between uses.
- CDIs are intended to be replaced as directed.

### Removing the e-module

1. Place the CCG containing the e-module flat on the table with the pocket facing you.
2. Slide your fingers under the e-module until it unsnaps from the CCG fabric.
3. Grasp the e-module while holding the end of the CCG to slide the e-module out of the CCG pocket.



The CCG should not be washed in a washing machine. Only use a clean and dry cloth to remove dust as needed.

### Storage instructions

The device is intended for use in a dry, room-temperature home setting. To avoid battery degradation, avoid long exposures to temperatures above 104° Fahrenheit (40° C). Store the device in a dry location.

### Instructions for safe disposal

Used CDIs and CCGs may be disposed of in normal trash bins. Consult your local laws regarding battery and electronic waste disposal.

# CHAPTER 4—WHEN TO USE YOUR DEVICE

## OVERVIEW

Your doctor will discuss the best times to use your device.

**It is important to use your device when you have RLS symptoms.**

- The NTX150 Tonic Motor Activation (TOMAC) System delivers 30-minute treatment sessions.
- Each session is designed to quickly relieve RLS symptoms.
- Relief typically begins soon after turning the device on and can last 2 hours after the end of the 30-minute session.<sup>2,3</sup>
- The device is not designed to prevent symptoms before they start.
- It is recommended that the device be used for treating symptoms after they start but before they become severe.

**Therefore, it is important to start a therapy session shortly after the start of RLS symptoms.**

**The device can deliver at least one and often two full sessions on a full charge. Therefore, it is important to ensure the device is fully charged at times of day that would have the most impact on your quality of life. These could include:**

- Bedtime, to help with sleep onset.
- Middle of the night, to help fall back asleep.
- Other times of day when RLS symptoms are severe.

**If used with the proper timing, clinical data suggests that use of NTX150 TOMAC System results in the following potential benefits:**

1. Reduction in RLS symptoms<sup>3</sup>
2. Improved sleep<sup>1</sup>

**Long-term use of NTX150 TOMAC System at proper times of day can also result in fewer days per week with RLS symptoms.<sup>4</sup>**

**If used with improper timing, the following may occur:**

- Low battery. If the device is used at times when not needed and not fully recharged, the battery charge may not allow for a full session when needed.

### General timing instructions

- Run a session after RLS symptoms start and before symptoms become severe.
- Always run a full 30-minute session. Devices will turn off by themselves after a session.
- On nights without RLS symptoms, do not use the devices.
- Prioritize sessions where reduced RLS symptoms could improve sleep (i.e., bedtime or middle of the night).
- When a single 30-minute session does not relieve symptoms, you have the option of running a second 30-minute session.
- A full charge typically allows for 1-2 sessions of use, dependent on therapy intensity and other usage factors and may differ on each leg and on a night to night basis.
- You should charge both devices daily after each use and keep charging the devices until the next session (keep the devices on the charger).
- Consider raising the Treatment Level to level 4 or level 5 if RLS symptoms are more severe than usual.
- Consider decreasing the Treatment Level to level 2 or level 1 if RLS symptoms are mild.

## Examples of common times to use the device

### 1. Before bedtime

- **Run a 30-min session after your RLS symptoms start but before they become severe.**
  - Based on your history of RLS symptoms, this may be in the two hours before bedtime, but could be earlier or later on some nights.
  - On nights when you don't have RLS symptoms, don't use the devices.
  - Start at level 3; if level 3 does not give enough relief, you may try a higher setting (level 4 or level 5).
- If your symptoms remain at bedtime, run a second session at bedtime at a setting that allows you to go to sleep (e.g. level 3 or level 2).

### 2. At bedtime

- **Wear your devices to bed. As soon as your RLS symptoms start, run a 30-minute session** at a level that allows you to go to sleep (typically level 3 or level 2).
  - If level 3 does not give enough relief, you may try a higher setting (level 4 or level 5). If level 3 is too distracting to go to sleep, try a lower setting (level 2 or level 1).
  - On nights when you don't have RLS symptoms, don't use the devices.
- If RLS symptoms remain after the session ends, you have the option of a second session.

### 3. After falling asleep (in the middle of the night)

- **Wear your devices to bed** but do not run a session until you have RLS symptoms. You can also keep the devices next to your bed instead of wearing them to bed.
- **When you wake up with RLS symptoms**, run a 30-min session at a setting that lets you go back to sleep (typically level 3 or level 2).
  - If level 3 does not provide enough relief, you may use a higher setting (level 4 or level 5). If level 3 is too distracting to go to sleep, use a lower setting (level 2 or level 1).
  - On nights when you don't have RLS symptoms, don't use the devices.
- If RLS symptoms remain after the session ends, you have the option of a second session.

## CHAPTER 5—TREATMENT

Before you begin using the device on your own you will attend a Personalized Activation Session where a technician will work with you to find the best settings to reduce your RLS symptoms. The technician will program these settings into your device and you can then follow the instructions in this chapter to complete a treatment.

### Inserting the e-module

1. Lay the Compressive Conductive Garment (CCG) and e-module flat as shown.




Compressive Conductive Garment (CCG)



e-module

2. Hold the CCG along the seams. While holding the battery (circled), slide the e-module into the pocket on the CCG until the metal contacts of the e-module can be seen through the circular openings in the black material.

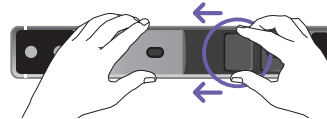

Metal Contacts

3. **Turn the band over so that the Nidra® logo is facing up.** Confirm the LED bar is aligned with the window in the CCG. Gently press down on the fabric all around the LED bar until you feel it snap into place.




LED Bar

### Placement of Charge-Dispersing Interface (CDI) on the device

1. If no CDI is on the device, or if you need to replace the CDI, choose a pouch from the box of CDI pouches.


2. Tear open the pouch at the top and remove one CDI.

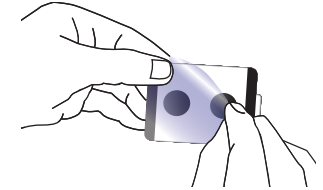

3. Grab the blue see-through disposable liner without the text by the tab, peel away the blue liner and discard.

4. You will see that the circular windows in the white foam backing of the CDI are exposed.

5. Line up these circular windows on the CDI with the metal contacts and firmly press the CDI onto the device.

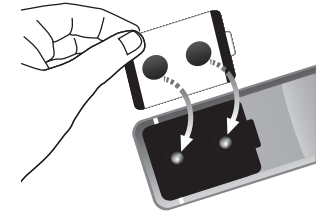

6. Remove the non-see-through reusable liner to expose the rectangular, sticky part of the CDI, and save the liner to protect the CDI after use.




7. Repeat steps 3–6 on the other device if (a) no CDI is present or (b) you need to replace the CDI.

## Finding the treatment target


### Treatment target

The nerve target for treatment curves around the top of the fibula (marked by the **BLUE** circle in the image). The **PURPLE** area indicates the ideal location of the CDI.

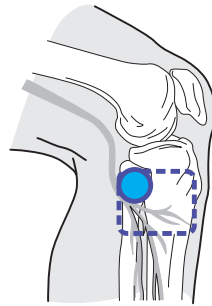


To find the treatment target (**BLUE** circle) sit with your **lower leg at a 90 degree angle to the floor**:

1. Identify the ankle bone on the outside of the ankle (**PURPLE** circle).
2. Move directly upwards along the outside of the leg (shown in **PURPLE**) until you reach the next bony landmark (**BLUE** circle).

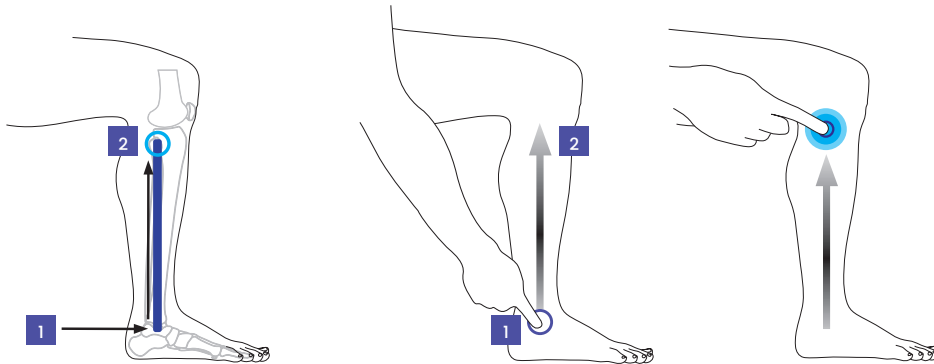


### Treatment target confirmation

The outside of the hamstring muscle attaches at the treatment target location (**BLUE** circle) and can be a useful way to confirm that you have found the treatment target. The **PURPLE** area indicates the position of the CDI and the hamstring attachment point is shown in **BLUE**.

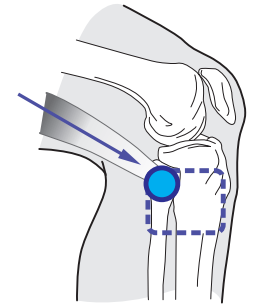

### Troubleshooting

If it is hard to find the bony landmark (**BLUE** circle from previous steps) try to rotate your leg (with your heel in place on the ground) from outside to inside. This will make it easier to see and feel the bony landmark. Repeat this movement on both legs to verify that the treatment target has been located.


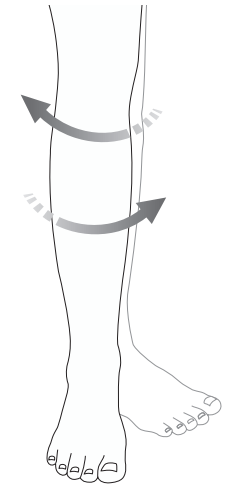


## CHAPTER 5—TREATMENT

### Putting on your devices


(images show the “R” [Right] Therapy Unit)

- Carefully peel the reusable liner from the Charge-Dispersing Interface (CDI) and save it to protect and reuse the CDI.
- Check that the skin is clean.
- Keep the liner on the CDI when not in use to extend the life of the CDI.
- To help keep your skin safe, change the CDI's every week. And if you live in a dry place, you can dampen your skin with a little water before starting therapy.

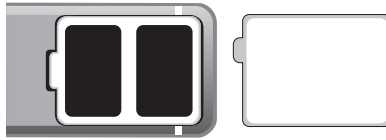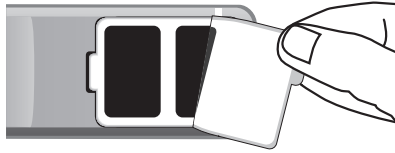

- The CDI contains two electrodes (the black rectangles). The electrode closest to the edge of the device is called the 'Edge Electrode'.
- The **BLUE** circle is where the edge electrode will meet up with the **treatment target**.

Head (“top seam”)

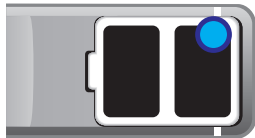

'Edge Electrode'

Feet (“bottom seam”)

#### IMPORTANT

Only turn on your Nidra devices when they are positioned over the treatment target. Do not turn them on prior to placement or remove them while therapy is active.


- Place the CDI directly on clean skin. Do not place the CDI over a skin wound or over a bandage or skin covering.




**WARNING!**

Do not place Charge-Dispersing Interfaces (CDIs) over open sores or an acute injury.

- Place the edge electrode – while attached to the device – so that the top edge corner (**BLUE** circle) covers the **treatment target** that was marked on your leg.

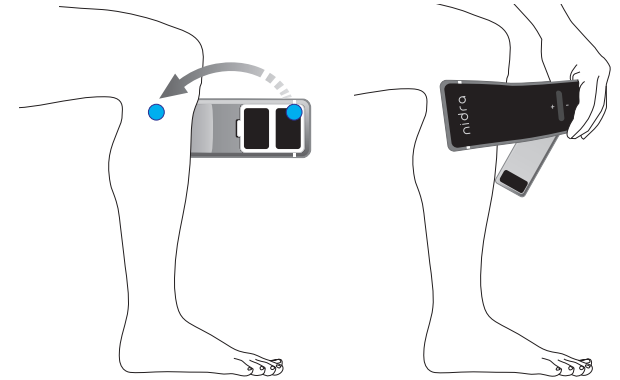


- Apply firm pressure around the CDI to ensure it sticks and stays in place.

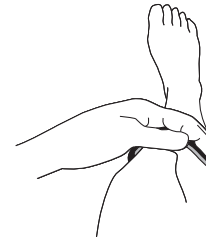


**Secure the device:** (images show the “L” [Left] Therapy Unit)

- Place one hand over the CDI to hold the device firmly in place.
- Pull the other end of the device and wrap it around the leg in the correct direction:
  - Left leg device wraps clockwise.
  - Right leg device wraps counterclockwise.
- Pull firmly to ensure a secure fit.
- Secure the device with velcro.

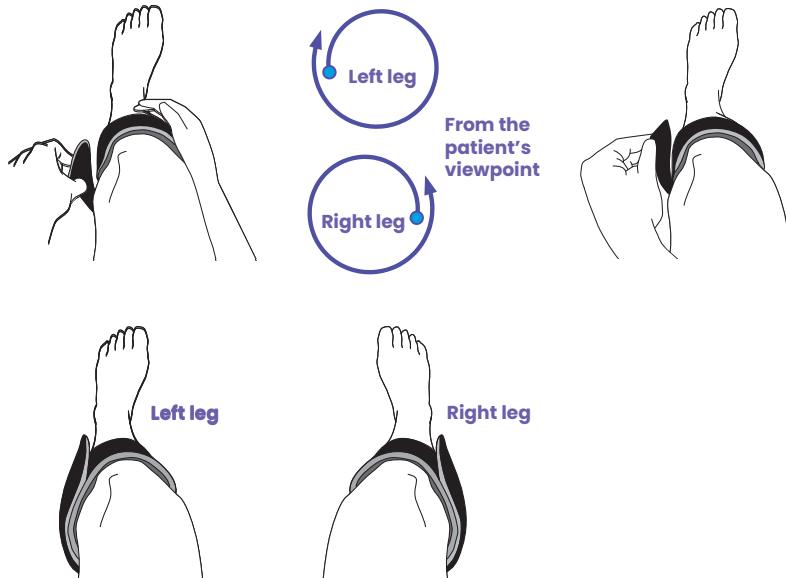

This is how the device will look after you are done. Depending on leg size, the position of edge of the device will vary.

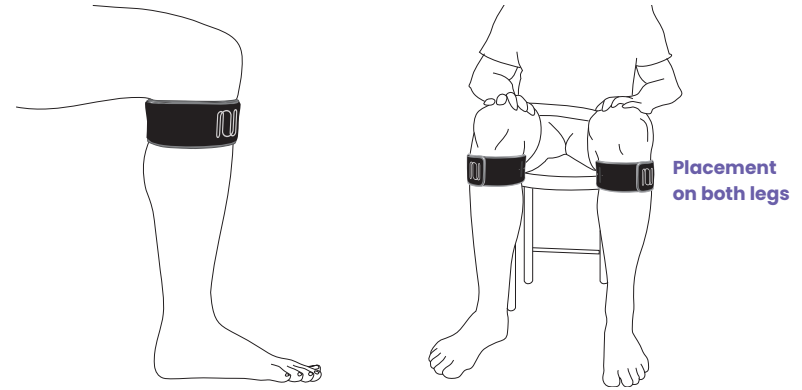


# CHAPTER 5—TREATMENT

## Starting a treatment

1. Check that devices for both legs have enough charge by checking for the **GREEN** Status Light while charging.
2. Remove the devices from the charging cables.
3. Verify the battery status by clicking the (+) button while the device is disconnected from the USB charging cable.
  - The Status Light will indicate charge status:
    - Solid **GREEN**: Typically enough charge for a full session.
    - Solid **ORANGE**: Typically enough charge for half a session.
    - Blinking **ORANGE**: Very low charge. A session cannot start.
4. Put the **LEFT** device on the left leg and the **RIGHT** device on the right leg.
  - If there is no Charge-Dispersing Interface (CDI) on the device, see the “Placement of CDIs on the Device” instructions at the beginning of this chapter.
5. Once devices are attached to both legs:
  - Start a session for each device by pressing and holding the (+) button for three seconds. The Status Light will indicate charge status while holding the (+) button.
    - a. From zero to 20 seconds, you will see blinking white Treatment Level Lights and a blinking **GREEN** Status Light as the intensity gradually increases to the mid-point of your calibrated range.
    - b. From 20 to 30 seconds, the Treatment Level Lights will show the therapy level of the device. During this 10 second period you will be able to set your treatment level from 1 to 5 within your range.

c. After 30 seconds, you can change your treatment level, but you will need to unlock the controls first;

- 1) If no white lights are lit, unlock the device by clicking (+) and (-) buttons at the same time and quickly – do not hold the buttons down.
- 2) The white lights that match the Treatment Level Lights will appear for 10 seconds while the device is unlocked.

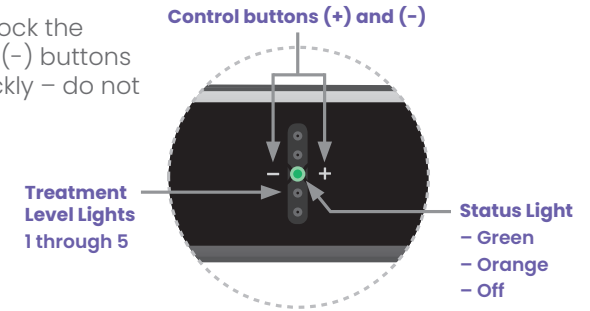

5. Relax and continue with your normal activity during the session, including falling asleep. The device will turn off by itself after 30 minutes.
6. If you fall asleep during the session or want to run another session later in the night, it's okay to continue wearing the devices after the session ends.
7. To remove the devices:
  - Click (+) and (-) at the same time briefly to unlock.
    - If white Treatment Level Lights appear, then the session is still in progress. Wait until the session is complete to remove the devices.
    - If no Treatment Level Lights appear, then the session is complete and you can remove the devices.
  - Detach the devices from your legs with the CDIs attached to the devices.
8. Replace the reusable liners on the CDIs.
9. After removal, connect the devices to the charging cable into the wall adapter.

## Adjusting treatment level during a treatment session

**Level 3 will be your most common treatment level. However, you may need to raise this level if your RLS symptoms are severe or lower this level if treatment at level 3 is distracting or causes discomfort.**

- If no white Treatment Level Lights are visible, unlock the device by clicking the (+) and (-) buttons at the same time and quickly – do not hold the buttons down. The white Treatment Level Lights that match the treatment level will appear and be visible for 10 seconds while the device is unlocked.
- Click the (+) button to raise treatment level by one level. Note that the number of Treatment Level Lights increases by one.
- Click the (-) button to decrease treatment level by one level. Note that the number of Treatment Level Lights decreases by one.

## Stopping a treatment

**You do not need to stop the session. The treatment will turn off by itself after 30 minutes and you can wear the devices through the night.**

- If treatment intensity is uncomfortable at any point, try decreasing the level to level 2 or level 1.
- If the intensity remains uncomfortable at level 1, you can stop treatment with the following steps:
  1. Press and hold the (-) button for 3 seconds to turn OFF device.
  2. As soon as the Treatment Level Lights turn off, release the (-) button and remove the device from your leg.
  3. Replace the reusable liner on each CDI.
  4. Charge the devices for future use.

## During a treatment session

**Relax and continue with your normal activity during the session, including falling asleep.**


- Treatment will turn off by itself after 30 minutes and you can wear the devices through the night.
- If the treatment level remains comfortable but RLS symptoms persist, try raising the treatment level.
- It is normal for the devices to feel warm; however if the devices feel uncomfortably warm, stop the session and contact Noctrix Customer Support at your earliest convenience.

## After a treatment session

**Click (+) and (-) buttons at the same time to confirm that the 30-minute treatment session is complete. If the white Treatment Level Lights appear, the treatment session is not complete and you should wait. If not, it is complete.**

### **DO NOT REMOVE THE DEVICES IF THEY ARE STILL POWERED ON.**

1. After verifying that the device is not powered on, unwrap the device from your leg.
2. Use your hand to hold down the skin next to the CDI. Gently peel off the CDI from your skin.
3. Replace the reusable liner on each CDI to keep them free of lint and dust.
4. Charge the devices for future use.

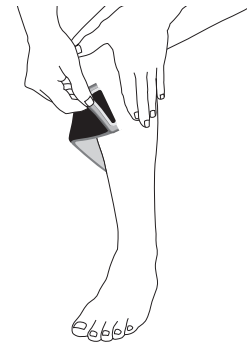

# CHAPTER 6—TROUBLESHOOTING

## PATIENT TROUBLESHOOTING

**For each of these scenarios try these steps in order until the problem is solved.**

### 1. The treatment sensation feels weaker:

1. Adjust the treatment to levels 4 and 5.
2. Stop therapy and change the location of the devices on your leg. Then, start therapy again.
3. Check that the device is fully charged and the lights show it is turning on and staying on.

If those steps do not work then change the CDIs. If this does not work contact Noctrix Customer Support to report the problem.

### 2. Device does not provide enough symptom relief

1. Unlock the device by pressing (+) and (-) buttons at the same time and then press the (+) button to raise level to level 4 or level 5 – to the highest level that is comfortable and not distracting.
2. If you waited until RLS symptoms were severe to start treatment, consider starting treatment earlier in the future.
3. Check placement of the device on your leg. If placement is incorrect, turn off the device, adjust the device placement, and restart the device.
4. If the problem remains, contact Noctrix Customer Support.

### 3. The treatment sensation is not comfortable

#### A. Treatment is consistently distracting or uncomfortable

1. Press the section of the device over the CDI firmly against your skin to improve contact.

2. Check placement of the device on your leg. If placement is incorrect, turn off the device, correct the device placement, and restart the device.
3. Adjust the intensity level:
  - If you feel pins/needles or tickling sensations, increase to level 4 or level 5.
  - Otherwise, lower the intensity to level 2 or level 1 using the (-) button until the sensation is comfortable.
4. If therapy is still distracting or uncomfortable at level 1, then power OFF device by pressing and holding (-) button for 3 seconds and then change the CDIs. If this does not work contact Noctrix Customer Support before your next use.

#### B. Temporary discomfort at the start of therapy

- Start at level 1 or 2 and raise to higher level after a few minutes.

#### C. Temporary discomfort during leg movements or in the middle of therapy

1. Press the section of the device over the CDI firmly against your skin to improve contact.
2. Check placement of the device on your leg. If placement is incorrect, turn off the device, correct the device placement, and restart the device.
3. Reduce movements that cause discomfort.
4. Contact Noctrix Customer Support before your next use.

### 4. The device gets in the way of sleeping

- If the size and shape of the device makes it difficult to sleep, then first check that the device is placed correctly on the leg. Also consider using the device just before bedtime instead of while trying to sleep.
- If the device feels too hot, then avoid tight clothing or bedding around the device (to allow air flow).

## 5. RLS symptoms get worse during or after treatment

- Are you using device when RLS symptoms are minimal to none? If so, then wait to use the device until symptoms are moderate or severe.
- Are you using device above level 3? If so, then use at level 3.
- Check CDI positioning and adjust if incorrect.
- Are you moving legs less than usual while wearing the device? If so, then move legs to a normal extent.
- Reduce to level 2. If still unresolved, then reduce to level 1. If still unresolved, then contact Noctrix Customer Support.

## 6. My device feels hot and is uncomfortable

- Press and hold (-) button for 3 seconds to turn OFF device.
- Contact Noctrix Customer Support before your next use.

## 7. Status Lights begin blinking during treatment

- Press and hold (-) button for 3 seconds to turn OFF device.
- Contact Noctrix Customer Support before your next use.

## 8. The CDI is drying up or losing its stickiness

The CDI should last approximately one week with typical use. If your CDIs are drying out faster than expected it may help to store them in a sealed bag when not in use.

For help with setting up, using, or caring for the NTX150 TOMAC System; or to report unexpected events, contact Noctrix Customer Support at [support@noctrixhealth.com](mailto:support@noctrixhealth.com), visit [www.nidrarls.com](http://www.nidrarls.com) or call **1-866-EASE-RLS (1-866-3273-757)**.

### Patent

This product may be covered by one or more of the following patents: US11103691B2, US11213681B2, US11872399B2, US11266836B2 and other US and international patents pending.

### References

1. Bogan RK, et al. Efficacy and safety of tonic motor activation (TOMAC) for medication-refractory restless legs syndrome: a randomized clinical trial. *Sleep*. 2023;46(10).
2. Data on file. CL-9, Rev1.0, Report, Quantifying duration of relief from a single 30-minute session of NTX100 TOMAC.
3. Buchfuhrer MJ, Baker FC, Haramandeeep S, et al. Noninvasive neuromodulation reduces symptoms of restless legs syndrome. *J Clin Sleep Med*. 2021;17(8):1685-1694.
4. Roy A, Ojile J, Kram J, et al. Long-term efficacy and safety of tonic motor activation for treatment of medication-refractory restless legs syndrome: A 24-Week Open-Label Extension Study. *Sleep*. 2023;46(10).

# APPENDIX A

## GUIDANCE AND MANUFACTURER'S DECLARATION

**Table 1**  
Guidance and Manufacturer's Declaration – Electromagnetic Emissions

The NTX150 Tonic Motor Activation (TOMAC) System is intended for use in the electromagnetic environment specified below. The customer or the user of the NTX150 Tonic Motor Activation (TOMAC) System should assure that it is used in such an environment.

Emissions test	Compliance	Comments
RF emissions CISPR 11	Group 1	NTX150 Tonic Motor Activation (TOMAC) System uses RF Energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emission IEC 61000-3-3	Complies	

**Table 2**  
Guidance and Manufacturer's Declaration – Electromagnetic Immunity

The NTX150 Tonic Motor Activation (TOMAC) System is intended for use in the electromagnetic environment specified below. The customer or the user of the NTX150 Tonic Motor Activation (TOMAC) System should assure that it is used in such an environment.

Test type	Compliance level	Electromagnetic environment – Guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input output lines	Mains power quality should be that of a typical household, commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2kV common mode	Mains power quality should be that of a typical household, commercial or hospital environment.
(50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical household, commercial or hospital environment.

**Table 2** (continued)  
**Guidance and Manufacturer's Declaration – Electromagnetic Immunity**

The NTX150 Tonic Motor Activation (TOMAC) System is intended for use in the electromagnetic environment specified below. The customer or the user of the NTX150 Tonic Motor Activation (TOMAC) System should assure that it is used in such an environment.

Test type	Compliance level	Electromagnetic environment – Guidance
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Voltage Dips 30% reduction, 25/30 periods at 0°	Mains power quality should be that of a typical household, commercial or hospital environment. If the user of NTX150 Tonic Motor Activation (TOMAC) System requires continued charging during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply.
	Voltage Dips >95% reduction, 0.5 period at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315°	
	Voltage Dips >95% reduction, 1 period at 0°	
	Voltage interruptions > 95% reduction, 250/300 periods	
Conducted RF IEC 61000-4-6	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the NTX150 Tonic Motor Activation (TOMAC) System, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of transmitter. Recommended separation distance = $1.2/\sqrt{P}$
Radiated RF IEC 61000-4-3	10 V/m	$d = 1.2/\sqrt{P}$ 80 MHz to 800 MHz $d = 2.3/\sqrt{P}$ 800 MHz to 2.7 GHz Where P is the maximum output power of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitter, as determined by an electromagnetic site survey* should be less than the compliance level in each frequency range**

\* Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which NTX150 Tonic Motor Activation (TOMAC) System is used exceeds the applicable RF compliance level above, NTX150 Tonic Motor Activation (TOMAC) System should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating device.

\*\* Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

## APPENDIX A

### GUIDANCE AND MANUFACTURER'S DECLARATION

**Table 2** (continued)  
**Immunity to RF Wireless Communications Equipment**

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum Power (W)	Distance (m)	Immunity Test Level (V/m)
385	380 - 390	TETRA 400	Pulse modulation 18Hz	1.8	0.3	27
450	430 - 470	GMRS 460 FRS 460	FM, ±5kHz deviation, 1kHz sine	2	0.3	28
710	704 - 787	LTE Band 13, 17	Pulse modulation 217 Hz	0.2	0.3	9
745						
780						
810	800 - 960	GSM 800/900 TETRA 800 iDEN 820 CDMA 850 LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
870						
930						
1720	1700 - 1990	GSM 1800 CDMA 1900 GSM 1900 DECT LTE Band 1, 3, 4, 25 UMTS	Pulse modulation 217 Hz	2	0.3	28
1845						
1970						

# APPENDIX B

## SYSTEM SPECIFICATIONS

**NTX150 Tonic Motor Activation (TOMAC) System Mass:** <150 g per Therapy Unit (<11 oz per two Therapy Units)

### Electrical Classification (Charging Accessories)

**Power Requirements:** Input 100-120 VAC, 60 Hz, 0.15-0.35A

**Replaceable Parts:** None

**Electrical Mains Adapters:** Only use Noctrix Health, Inc. provided electrical mains adapter for the Charger.

<b>Manufacturer:</b>	Globtek
<b>Manufacturer Model:</b>	GTM46161-155.0-USB2A
<b>Input:</b>	100-240 V AC, 0.45 A
<b>Output:</b>	5V, 3.2 A
<b>Certification:</b>	CB 60950, CB 60601-1, CB 62368
<b>Means of Protection:</b>	2 x MOPP

**Charging Cable:** Input: 5V DC  
Output: 5V DC

Isolation from Mains is accomplished by unplugging the Approved Charging Adapter.

### Electrical Classification (Therapy Units)

**Classification:** Internally Powered when delivering therapy; conforming to Class II when charging

**Electrical Isolation:** Type BF Applied Part. User is operator when charging.

**Ingress Protection:** IP20: NTX150 TOMAC System outside of carrying case  
IP22: NTX150 TOMAC System inside of carrying case

The factory-supplied CCG and drawstring carrying bag provide for the carrying case when IP22 ingress protection is needed.

**Internal Battery (not serviceable):** 3.7V 1000mAh

**Patient Leakage Current:** < 50 uA DC; < 500 uA AC

**Rated Duty Cycle:** 100% duty cycle square wave for 30-minute therapy session

<b>Waveform:</b>	Current Controlled, Charge-Balanced Square Wave
<b>Pulse Width:</b>	125 uS nominal
<b>Pulse Repetition Frequency:</b>	1,000 – 8,000 Hz; 4,000 Hz nominal
<b>Output Current:</b>	40mA maximum, current-controlled, physician-programmable
<b>Output Voltage:</b>	Current-controlled output depending on skin impedance; compliance voltage 60V peak
<b>Rated Load:</b>	The therapeutic waveform is current-controlled and showed to conform to the stated parameters over the load range of 200 – 1,200 Ohms impedances.

**Wireless Radio:** Contains FCC ID: XPYBMD380

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**Nominal service life:**

- Charge-Dispersing Interfaces: 1 week
- NTX150 TOMAC System, charger, charging cables and battery: 3 years

Operating, Storage, and Transit Conditions	Operating	Storage and Transit
<b>Temperature</b>	41 °F - 95 °F (5 °C - 35 °C)	14 °F - 104 °F (10 °C - 40 °C)
<b>Humidity</b>	5 - 95% Non-condensing RH	5 - 95% Non-condensing RH
<b>Pressure</b>	64 - 102 kPa (1 ATM)	59 - 102 kPa (8.6 - 15 psia)

© 2026 Noctrix Health Inc.  
Noctrix, Noctrix Health and Nidra  
are all registered trademarks  
of Noctrix Health, Inc.

